

STAMP - Quality Control (QC)

Did you forget?

November 12, 2013 a new quality control in STAMP was introduced. This new quality check retrieves all serological HLA types not found among the recipients own HLA, acceptable mismatches or identified antibodies. This new STAMP check is an additional quality control and is meant as a help and it does not inflict with the activation of patients to STAMP.

The new QC is accessible from two places in the STAMP screen from 'STAMP check' and 'Acc. Mismatches'.

Entry from 'STAMP check'

The screenshot shows the STAMP software interface. At the top, there is a menu bar with 'File', 'Edit', 'View', 'Window', and 'Help'. Below the menu bar is a toolbar with various icons. The main window displays the following information:

```
STAMP - ACCEPTABLE MISMATCHES PROGRAMME 9201
Scandia Number          Pers Num          Sex M Male          ABO AB
Surname                  cmo 29-NOV-2011 test  First name      cmo 29-NOV-2011 test S
Tx Center                CP København          Country         DK Denmark
Curr wait list           KI Kidney            Wl status       N Wait for tx number 1
Date of wl entry         30-NOV-2011          Dial/Tr cn     _
Narr eq ser HLA         A 1;B 13,7;Cw 2,3;DR 1;DQ 9,1;
Repea mismatches
Most recent PRA 1        30-MAY-2012 I% 60          II% 60
Most recent PRA 2        I%          II%
Identif antib           Cw3;
Date of accept           07-MAR-2012          Sign up Y Yes Status A Active
Methods of determ        _
Acc. mismatches          _ A2,3;B5,8;Cw3;DQ2;DR2,3
Comments
STAMP check               Event log _ Calc. frequency _ Perm. withdrawal _
Date/User of First Reg/Last Change 07-MAR-2012 SCTP _ 14-JUN-2012 SCTP
```

At the bottom of the window, there is a status bar that reads: '* POP UP * Count: *1 <Replace>'.

When you enter the pop-up by pressing >F7< you will see a display of all executed STAMP QC's.

The new QC is found in the bottom, listing all antigens not given as own HLA antigens, acceptable mismatches or identified antibodies.

File Edit View Window Help

Quick Connect Profiles

STAMP - ACCEPTABLE MISMATCHES PROGRAMME 9201

Scandia Number _____ Pers Num _____ Sex M Male _____ ABO A

Surname _____ First name _____

STAMP Quality Control

<input checked="" type="checkbox"/> Wait time > 1 yr or child	Age: 52 years	Wait time: 68 months	<u>Yes</u>
HI PRA >= 80% at present		HI Highly immunized	<u>Yes</u>
Most recent PRA < 3 months		20-JAN-2011 33 months 23 days	<u>No</u>
HI >= x2 > 3 months		samples over 18 months 0 days	<u>Yes</u>
A,B,Cw,DR,DQ splits	A 2,*28; B 27,37; Cw 10,9; DR 10,11; DQ 5,7;		<u>No</u>
Acc. mm. splits		All acc. mismatches are splits	<u>Yes</u>
Id. antib. splits		DR *2;	<u>No</u>
No identif. AB in Acc. mm.		Cw 1;	<u>No</u>
Consistent Sex/Cow/HLA		Consistent Sex/Cow/HLA	<u>Yes</u>
All HLA checked	A23; B58; Cw15,18,2,4,5,6; DR18;		<u>No</u>

Date/User of First Reg/Last Change 11-FEB-2010 _____ 18-JUN-2012 _____

Count: *0 <Replace>

A 'Yes' in this QC is not mandatory to get the recipients accepted to STAMP, it is merely a help to see if you have evaluated all HLA antigens.

Entry from 'Acc. Mismatches'

File Edit View Window Help

Quick Connect Profiles

STAMP - ACCEPTABLE MISMATCHES PROGRAMME 9201

Scandia Number 13 Pers Num _____ Sex M Male _____ ABO A

Surname _____ First name _____

Tx Center _____ Country S Sweden _____

Curr wait list KI Kidney Wl status N Wait for tx number _____

Date of wl entry 19-FEB-2008 Dial/Tr cn AH SUNDSVALL, Dialysavd

Narr eq ser HLA A 2,28;B 27,37;Cw 10,9;DR 10,11;DQ 7,5;

Repea mismatches A3;B15,8;DR2,4,6

Most recent PRA 1 20-JAN-2011 I% 97 II% 91

Most recent PRA 2 21-OCT-2010 I% 99 II% 93

Identif antib A1,11,24,29,3,30,31,34,36,80; B13,18,27,35,39,41,42,45,46

Date of accept 11-FEB-2010 Sign up Y Yes Status A Active

Methods of determ CDC, FLOW

Acc. mismatches A25,26,32,33,43,66,68,69,74;B38,44,47,59;Cw1;DQ7;DR13,14

Comments _____


STAMP check _____ Event log _____ Calc. frequency _____ * Perm. withdrawal _____

Date/User of First Reg/Last Change 11-FEB-2010 _____ 18-JUN-2012 _____

* POP UP *


Count: *1 <Replace>

Enter the 'Acc. Mismatches' pop-up by pressing >F7<

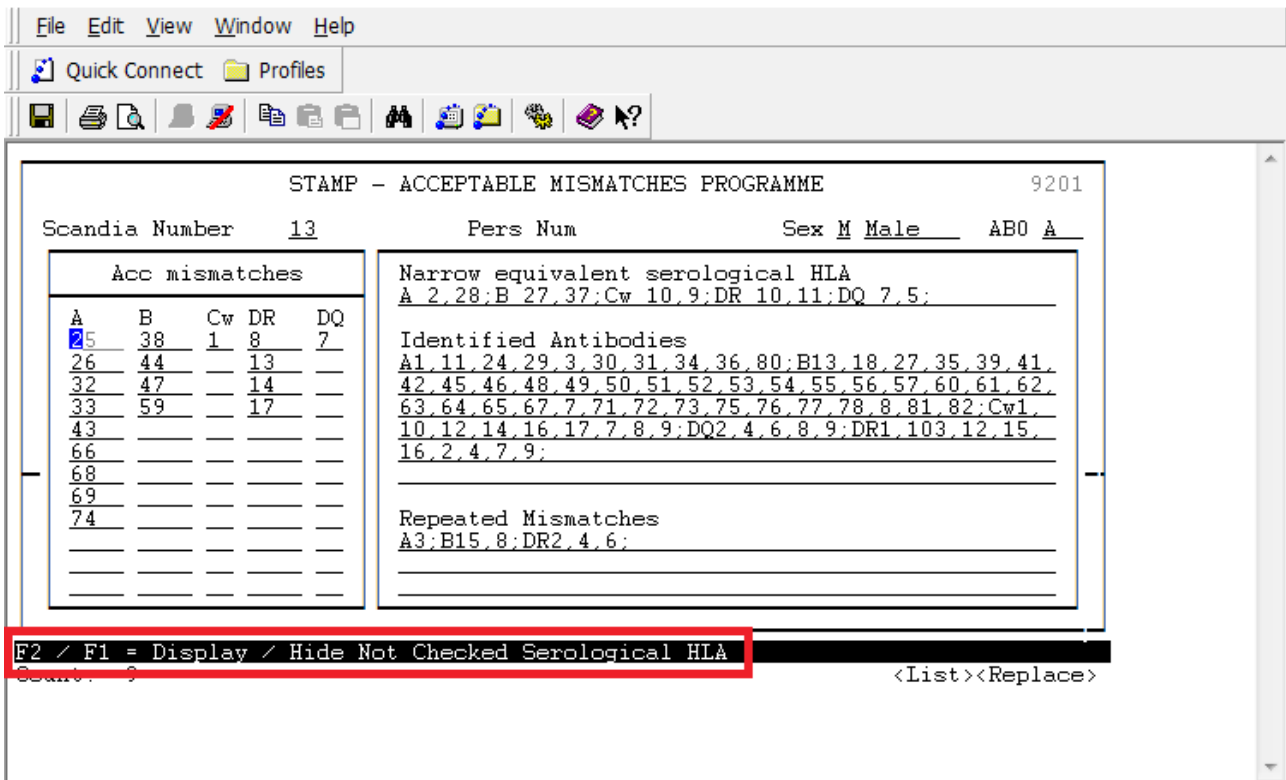

By pressing >F2< all antigens not given as own HLA antigens, acceptable mismatches or identified antibodies will be display in the right bottom corner.

The box 'Not Checked serological HLA' does not need to be empty to get the recipients accepted to STAMP, it is merely a help to see if you have evaluated all HLA antigens.

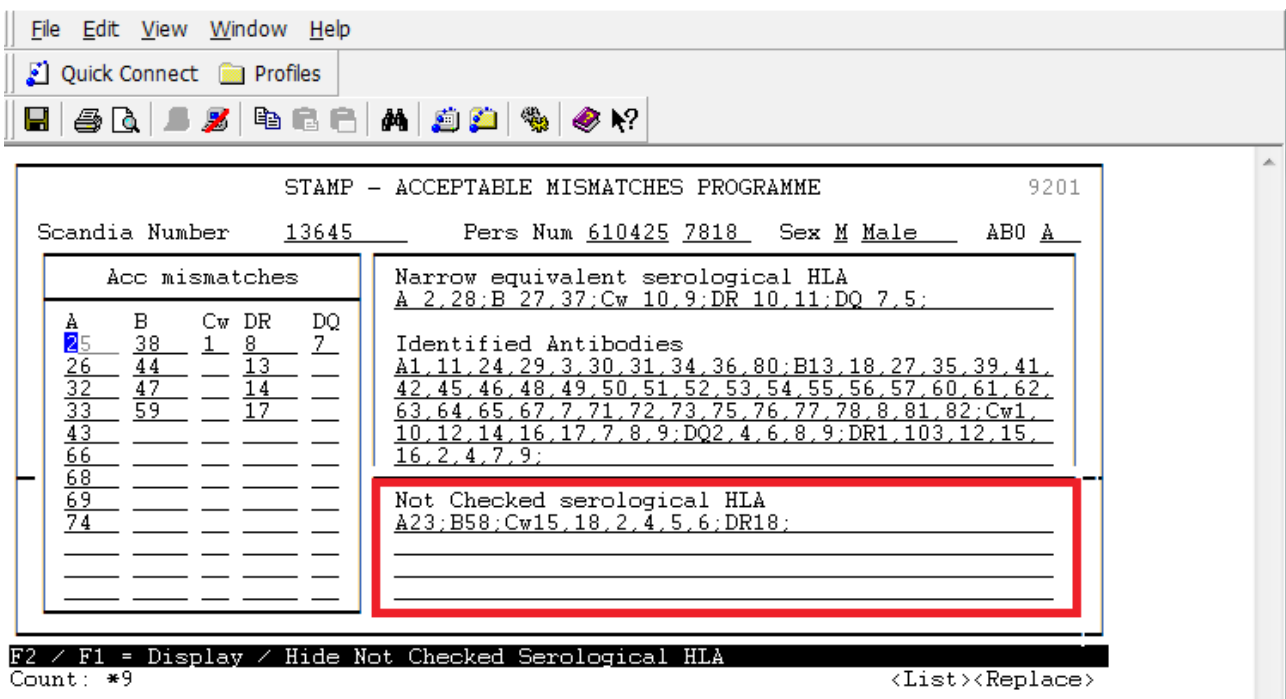

Pressing >F1< will hide the 'Not checked serological HLA' box again.